



ToxWipe™ 7 - Saliva Drug Test Kit

| Specifications | |
|--------------------|---|
| Product | ToxWipe™ saliva drug test |
| Detection: | Detect AMP, MET, COC, OPI, OXY, BZO & THC in saliva in 1 test. |
| Content: | Per Pack - 1 x ToxWipe Test - 1 x Buffer Cap - 1 x Silica Pack |
| Compliance: | AS/NZS4760:2019 |

Product Features

- Collect saliva and read results for 7 types of drugs from the same device.
- Intuitive to use and no need for shaking, squeezing, or pressing.
- The ToxWipe™ will give reliable results without 5 minutes

Direction of Use

Step 1: Sample collection:

DO NOT eat or drink 15minutes prior to testing. Place test device (without buffer cap) into mouth, swab under tongue and inside cheeks, as salivary glands under tongue and inner upper cheek area is the most effective way to collect saliva sample. * Depending on patient and possible dry-mouth conditions, collection time may vary around 1 minutes. Please make sure test tip have collected enough saliva sample, as insufficient test sample may result in invalid test results.

Step 2: Collection Verification

It is very easy to tell if enough sample have been obtained; you will see a Blue Line within both test windows appear, and start to travel up ("wicking" blue lines)

Step 3: Reading Result

Insert pad (tip) of collection device into the buffer cap provided. Place test cassette flat on table, wait 5 min. before reading result. The presence of a line in T (test result window) means Negative result. The absence of a line in T means Positive result. Absence of line in C (control window) means test is Invalid.

AMP (Amphetamine)
COC (Cocaine)
BZO (Benzodiazepines)
MET (Methamphetamine)
OPI (Opiates)
THC (Marijuana)
OXY (Oxycodone)

